## EN Combined technique: versatility and comfort in the non-vital teeth whitening

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### QUEJA

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### EN CLINICAL/RADIOGRAPHIC EXAM

All teeth were healthy, but yellowish.

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### TREATMENT

The combined technique was recommended, with one in-office application of Whiteness HP Blue 35% for 45 minutes and 2 weeks of at-home use of Whiteness Perfect 10% in the trays for 6 to 8 hours a day (during the night). The patient was very pleased with the results and had no complaints regarding hypersensitivity during the treatment.

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STEP BY STEP | PASO A PASO

1a and 1b. Teeth with shade similar to A3 for incisors and A3.5 for canines.

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3. Results after the whitening session with a noticeable change in shade.

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4. Adaptation of the tray for at-home use of whitening gel for 2 weeks.

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// Whiteness HP Blue 35% with calcium is a product that, due to its pH stability, has a slow and steady release, reducing, this way, the possibility of causing sensitivity. It can be used in a single application with no need for exchanging the gel. The calcium allows for less mineral loss of the enamel compared to those products without calcium. The combined technique is a very interesting alternative that combines the benefits of the in-office technique (quick whitening) with the at-home one (longer shade stability) in a short period of time (15 days). //

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